oto-fresh®

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Discover our oto-fresh-products for optimal cleaning and care of your bachmaier® hearing protectors! Available at your hearing aid audiologists.  
// easy to handle // long-lasting effect // optimal care

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## EC Type-Examination

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
The hearing protection CUSTOMFIT bachmaier® *fidelity* was tested in accordance with standard EN 352-2: 2002 and is certified and controlled in accordance with PPE Regulation (EU) 2016/425 by PZT GmbH, Bismarckstr. 264 B, 26389 Wilhelmshaven, Germany.

*The manufacturer declares under his sole responsibility the conformity of the product in accordance to PPE Regulation (EU) 2016/425.*

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Your specialist for custom-made hearing protection:

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# CUSTOMFIT bachmaier® fidelity

Clear and accurate sound.

When used properly, this custom-made hearing protection product protects against dangerous or harmful noise. The bachmaier® fidelity hearing protection provides a clear communication - even in very noisy surroundings. The special technology combines distortion-free sound with excellent protection. Hearing is thus protected from excessive noise exposure while speech intelligibility is affected only imperceptibly.

## Special areas of use

// concerts // travel // teaching professions // craft and workshops // clean industry

## Correct use

The hearing protection is worn in the auditory canal and, when fitted correctly, protects against harmful noise. Any other use is not as intended and therefore not permissible.

Take the handle or surround between thumb and forefinger and, using light pressure, twist the hearing protection into the ear until it sits comfortably.

**IMPORTANT: The protection with the red mark or the R before the serial number is for the right ear!**

## Usage Instructions

This hearing protector will only provide its full protective function when worn at all times during noise exposure. If this advice is not followed then protection can be seriously impaired. Have the fit of the protection checked regularly by your audiologist as changes in the auditory canal can make the seal less effective. In this case, the hearing protection should be renewed.

The user should ensure that the hearing protection is correctly inserted and tested as stipulated in the manufacturer's instructions, and is regularly examined (at least every 6 months) for the need to be tested.

**WARNING: The ear plugs with a grip should not be used with cord when there is a danger that the drawstring could get snagged or caught in moving machinery. In this case we recommend removing the cord.**

## Storage and Care

Store your hearing protection in the case supplied at +55°C max. Clean the CUSTOMFIT bachmaier® fidelity after each use. Remove any ear wax residue using the pin provided.

**IMPORTANT: The attenuation filter must not come into contact with moisture. Before cleansing the product, it should be removed from the filter holder.**

*This product may be susceptible to certain chemical substances.*

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\* When inserting the filter b10 into the individually moulded earplug the requirement for minimum sound attenuation acc. to EN 352-2:2002 is not fulfilled.

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8270 | 0822 - 7 | ©bachmaier®