CUSTOMFIT **bachmaier**[®] *silence mini*

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CUSTOMFIT **bachmaier**[®] *silence mini*

Noise protection with special shaping.

The possible uses of the CUSTOMFIT **bachmaier**[®] *silence mini* are highly diverse and your hearing is reliably protected from irreparable damage.

Special areas of use

// library // tram // when traveling // family or household noise

Correct use

Take the handle between thumb and forefinger and, using light pressure, twist the hearing protection into the ear until it sits comfortably.

IMPORTANT: The protection with the red mark or the R before the serial number is for the right ear!

Usage Instructions

This ear protector will only provide its full protective function when worn at all times during noise exposure. If this advice is not followed then protection can be seriously impaired. Have the fit of the protection checked regularly by your specialist as changes in the auditory canal can make the seal less effective.

In this case, the hearing protection should be renewed.

Storage and Care

Store your noise protection in the case supplied. Clean the CUSTOMFIT **bachmaier**[®] *silence mini* after each use.

Cleaning and Care

Discover our **oto-fresh**-products for optimal cleaning and care of your **bachmaier**[®] noise protectors! Available at your hearing aid audiologists.

// easy to handle // long-lasting effect // optimal care

This product may be susceptible to certain chemical substances.