# CUSTOMFIT **bachmaier**<sup>®</sup> *sleep*

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# CUSTOMFIT **bachmaier**<sup>®</sup> *sleep*

Pleasantly quiet without being completely cut out.

We are pleased that you have done something good for your hearing!  
The noise protection CUSTOMFIT **bachmaier**<sup>®</sup> *sleep* reduces ambient noise to a comfortable level but without completely eliminating all sounds. For safety reason it is important to be able to hear significant sounds - e.g. a ringing telephone or siren.

## Special areas of use

// noise-related problems in falling and staying asleep  
// noisy sleep surroundings // when traveling

## Correct use

Take the handle between thumb and forefinger and, using light pressure, twist the hearing protection into the ear until it sits comfortably.

**IMPORTANT: The protection with the red mark or the R before the serial number is for the right ear!**

## Usage Instructions

This noise protector will only provide its full protective function when worn at all times during noise exposure. Have the fit of the protection checked regularly by your specialist as changes in the auditory canal can make the seal less effective. In this case, the hearing protection should be renewed.

## Storage and Care

Store your hearing protection in the case supplied. Clean the CUSTOMFIT **bachmaier**<sup>®</sup> *sleep* after each use.

## Cleaning and Care

Discover our **oto-fresh**-products for optimal cleaning and care of your **bachmaier**<sup>®</sup> noise protectors! Available at your hearing aid audiologists.

// easy to handle // long-lasting effect // optimal care

*This product may be susceptible to certain chemical substances.*