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# CUSTOMFIT bachmaier® splash

For sheer fun in the water.

We are pleased that you have done something good for your hearing. This individual ear protection CUSTOMFIT bachmaier® splash reduces the danger of infection which is particularly great when swimming and showering.

## Special areas of use

// showering // bathing // swimming // splashing around

## Correct use

Take the the lug between thumb and forefinger and, using light pressure, twist the hearing protection into the ear until it sits comfortably.

**IMPORTANT: The protection with the red mark or the R before the serial number is for the right ear! This earplugs are no protection while diving!**

## Usage Instructions

This ear protection only achieve their protective effect if it sits correctly. Have the fit of the protection checked regularly by your specialist. Changes in the auditory canal can lead to a deterioration in the fit. In this case the noise protection should be replaced.

## Storage and Care

Store your swimming ear protection in the case provided and clean the CUSTOMFIT bachmaier® splash after each use.

## Cleaning and Care

Discover our oto-fresh-products for optimal cleaning and care of your bachmaier® hearing protectors! Available at your hearing aid audiologists.

// easy to handle // long-lasting effect // optimal care

*This product may be susceptible to certain chemical substances.*