oto-fresh®

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Discover our **oto-fresh**-products for optimal cleaning and care of your **bachmaier**® hearing protectors! Available at your hearing aid audiologists.
// easy to handle // long-lasting effect // optimal care

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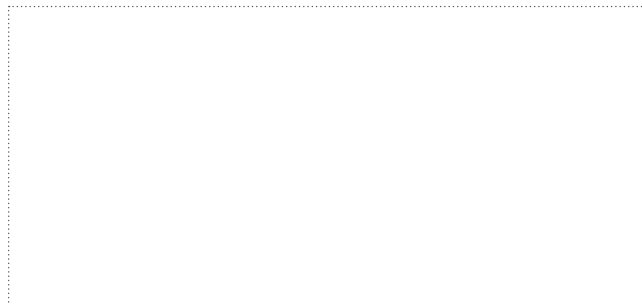
EC Type-Examination

The hearing protection **CUSTOMFIT bachmaier**® *work soft* was tested in accordance with standard EN 352-2: 2002 and is certified and controlled in accordance with PPE Regulation (EU) 2016/425 by PZT GmbH, Bismarckstr. 264 B, 26389 Wilhelmshaven, Germany.

*The manufacturer declares under his sole responsibility the conformity of the product in accordance to PPE Regulation (EU) 2016/425.
Find the declaration of conformity for download under:
www.bachmaier.de/ke*

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Your specialist for custom-made hearing protection:

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Discover all **CUSTOMFIT** hearing and noise protection at www.bachmaier.de

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CUSTOMFIT bachmaier® work (soft)

High attenuation and optimal signal perception.

The hearing protection CUSTOMFIT bachmaier® work soft was developed to protect the hearing from dangerous or harmful noise, if used properly. Harmful noise is reduced to ensure a continuing awareness of warning signals. The intelligibility of speech is barely compromised.

Special areas of use

// construction sector // agriculture and forestry // industry // the trades

Correct use

The hearing protection is worn in the auditory canal and, when fitted correctly, protects against harmful noise. Any other use is not as intended and therefore not permissible.

Take the handle between thumb and forefinger and, using light pressure, twist the hearing protector into the auditory canal until it sits comfortably.

IMPORTANT: The protector with the red mark or the R before the serial number is for the right ear!

Usage Instructions

This hearing protector will only provide its full protective function when worn at all times during noise exposure. If this advice is not followed then protection can be seriously impaired. Have the fit and functioning of the protection checked regularly by your audiologist since damage or changes in the auditory canal can make the seal less effective. In this case, the hearing protection should be renewed.

The user should ensure that the hearing protection is correctly inserted and tested as stipulated in the manufacturer's instructions, and is regularly examined (at least every 6 months) for the need to be tested.

WARNING: The ear plugs should not be worn with a cord when there is a danger that the drawstring could get snagged or caught in moving machinery. In this case we recommend removing the drawstring.

Storage and Care

Store your hearing protection in the case supplied at +55°C max. Clean the CUSTOMFIT bachmaier® work soft after each use. Remove any ear wax residue using the pin provided.

IMPORTANT: Should the filter become wet, carefully blow moisture out!

This product may be susceptible to certain chemical substances.

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8246 | 0822 - 7 | ©bachmaier®